

BladeBOX

100 | Scalpel Blade Removal System

**Work Safe Today,
Play Safe Tomorrow**



www.qlicksmart.com

 **QLICKSMART®**
... and be safe

The forgotten sharps injury

Scalpel injuries are up to 6 times more likely to occur than needlestick injuries. They are particularly common in surgical settings. Regrettably, healthcare organisations and personnel have underestimated the likelihood of these injuries.


Scalpel cuts can result in infections, blood-borne diseases and psychological distress. Serious injuries could require microsurgery to repair a digital artery, nerve and/or tendon.

Many such scalpel injuries can be prevented using the Qlicksmart **BladeBOX**.

With single-handed activation, the BladeBOX facilitates compliance with international regulatory requirements, protecting workers against health hazards related to bloodborne pathogens and other injuries.



BladeBOX allows all scalpel blade users to maintain their preferences for the standard metal handles.

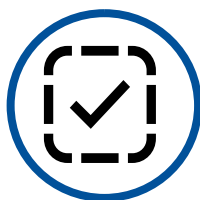

BladeBOX is simple to use with a quick “push, click, and pull” action to remove and contain used scalpel blades.



This easy-to-use solution provides a lower cost alternative to safety scalpels.



Used together with a hands-free passing technique, the BladeBOX can prevent up to 5 times more injuries than a safety scalpel.



Qlicksmart's scalpel blade removers have won multiple awards, which demonstrate its practicality and effectiveness in reducing scalpel injuries.



The compact size of the BladeBOX makes it an ideal economical and space-saving option for smaller work areas with limited storage space.






Specifications

Colour Versions	Yellow
Product Code	QBB2Y
GMDN code	46235
Blade capacity	100 Blades - the BladeBOX shuts off automatically when maximum capacity is reached.
Compatible Blades & Handles	Handles: #3, 3L, 4, 4L, 7, 9, Lawton long handle. Blades: #10, 10a, 11, 12, 13, etc. up to and including #25a, 27, and 36.
Incompatible Blades & Handles*	Barron handle, Beaver handle, disposable handles, thick and round bulbous handles, ergonomic handles
Size (per unit)	Length 119mm ,Width 100mm , Height 106mm
Weight (per unit)	Approximately 150g.
Accessories	Mounting Bracket - the bracket holds the BladeBOX in place to allow single-handed removal of scalpel blades.
Material	Body: Acrylonitrile butadiene styrene (ABS) Inner mechanism: Polyacetal (POM)

*The new BladeFLASK EVO is compatible with Barron handles, thick and round bulbous handles, and ergonomic handles.


Latex	No
DEHP	No
Sterile	No
Multi-use	Each unit removes 100 blades before disposal. Do not attempt to empty contents.
Disposal Method	Meets AS 3825:2020 Standard (Procedures and devices for the removal, containment and disposal of scalpel blades from scalpel handles.) It is certified with UN Transport requirements for dangerous goods. The infectious substance logo on the front label and the correct sharps container colour of the unit allow it to be disposed of as medical waste. Please follow your establishment's procedures for the disposal of medical waste. To be disposed of as a single-use sharps container.
Regulatory Information	Australia - Listed under TGA ARTG NO: 146821. Europe - Class I medical device under EU MDR 2017/745 and CE marked. UK - Class I medical device under UK MDR 2002 and UKCA marked. ISO 23907-1:2019 Compliant (Sharps injury protection - requirements and test methods - Part 1 single-use sharps containers).

How to use BladeBOX

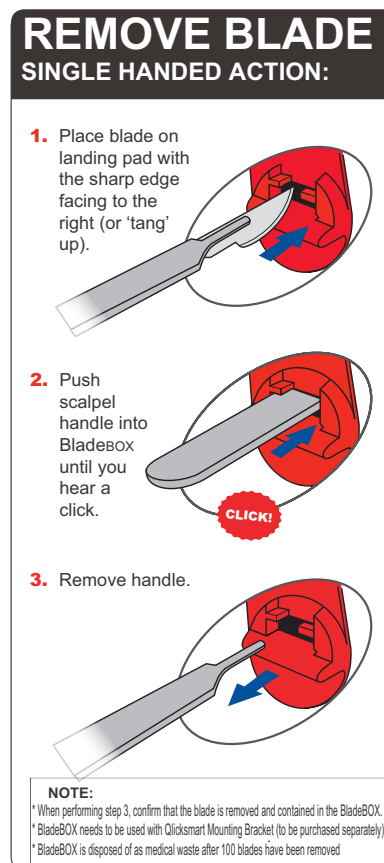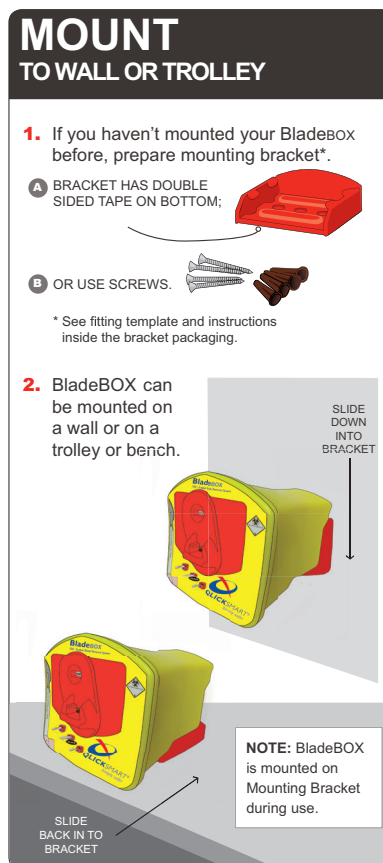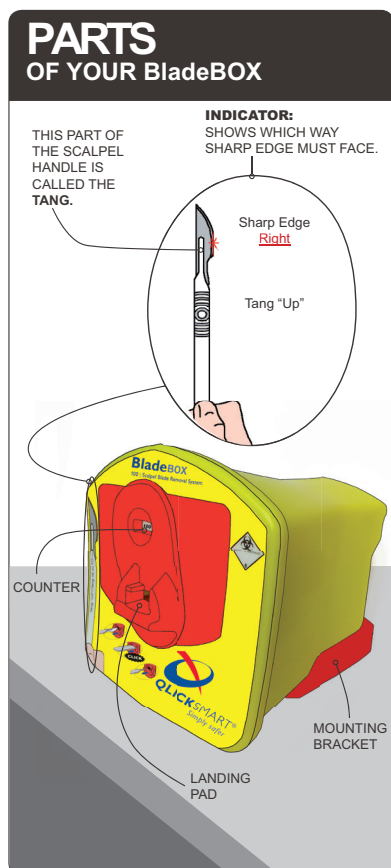

Other Qlicksmart Products


BladeFlask
Scalpel Blade Remover

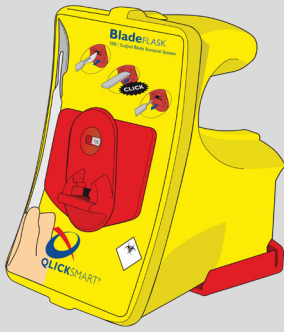

BladeFlask EVO
Scalpel Blade Remover




BladeCASSETTE Sterile
Scalpel Blade Remover

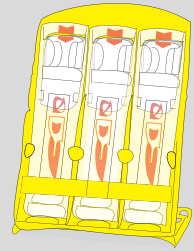

SnapIT and SnapIT Lite Ampoule Openers



BladeNeedleSYSTEM Sterile 2-in-1
Needle Counter + Blade Remover

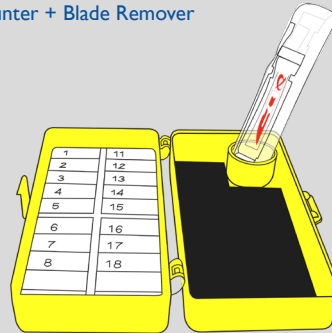

CheckCLIP Medication Identification



About Qlicksmart

Qlicksmart Pty Ltd is committed to making healthcare professional's lives easier and safer, so that they can focus on patient care. Established in 1999 by two Australian physicians to combat scalpel cuts in hospitals, our award-winning safety devices are now being used in over 50 countries worldwide. Qlicksmart works with innovators from the healthcare sector to deliver solutions which target the overlooked concerns of nurses, surgeons, and healthcare facilities. To learn more about Qlicksmart's safety devices, please visit www.qlicksmart.com or ask your distributor for a demonstration today.

Qlicksmart Pty Ltd.



Level 1, 148 Boundary Street
West End, Brisbane
QLD, Australia 4101



+61 7 3844 1182



hello@qlicksmart.com
www.qlicksmart.com



Search "qlicksmart"
and stay connected



Scan the QR
Code to learn
more about
QLICKSMART



Distributed by: